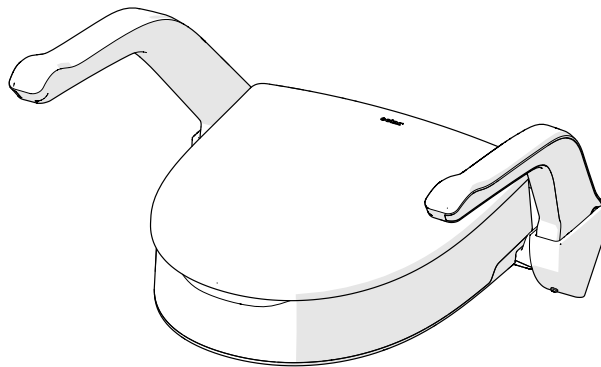
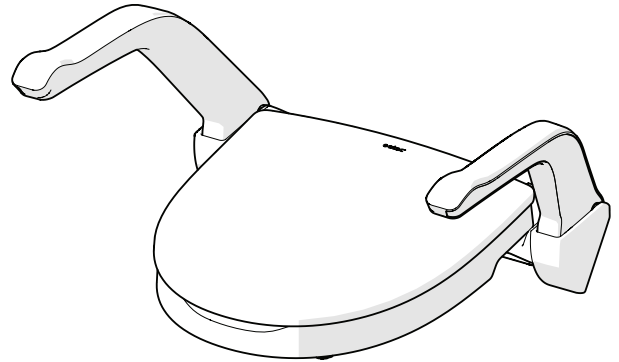
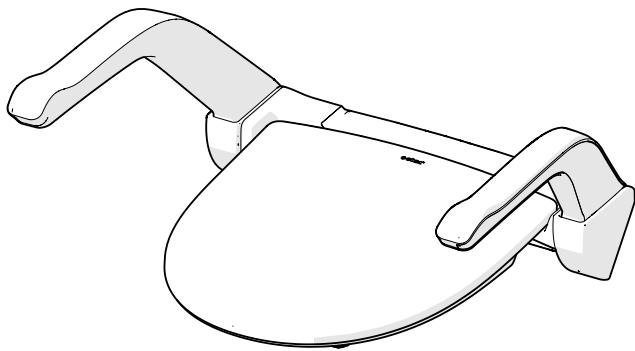
# User manual

## My-Loo Fixed

**etac**<sup>®</sup>  
Creating Possibilities

78682 - 2025-03-03 - Version 11


sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

[www.etac.com](http://www.etac.com)

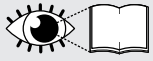MD CE


A

8030 1510


8030 1515

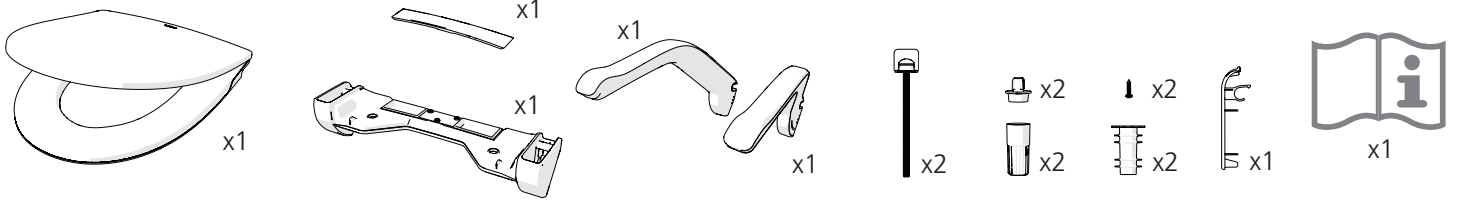1


8030 1511

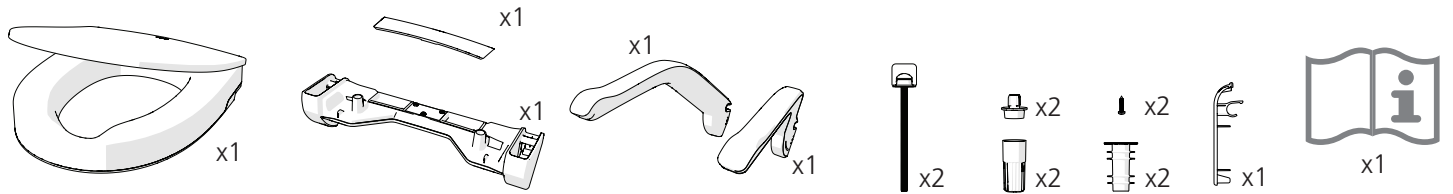8030 1516


8030 1512

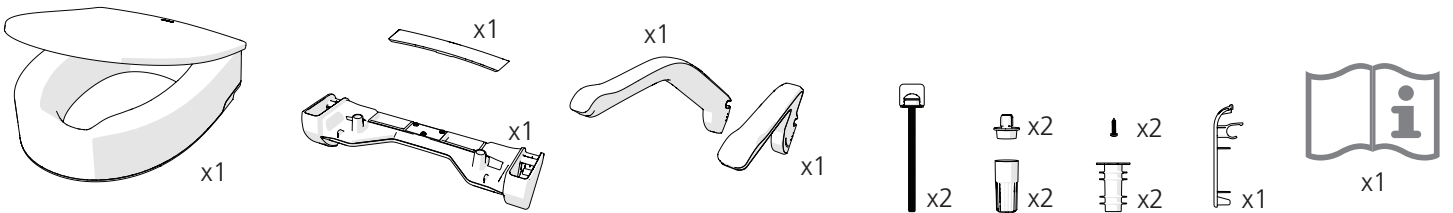8030 1517


8030 1513

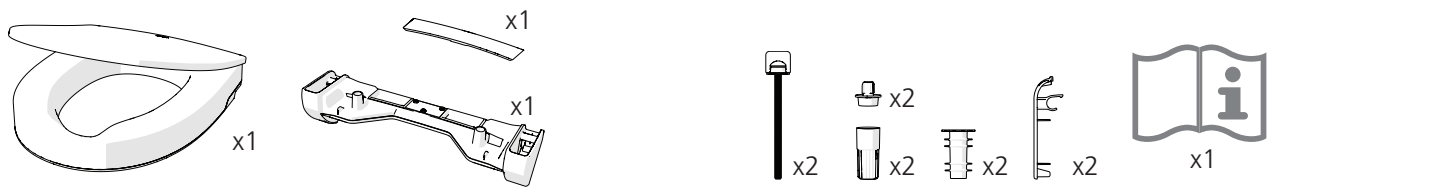
8030 1518

8030 1514

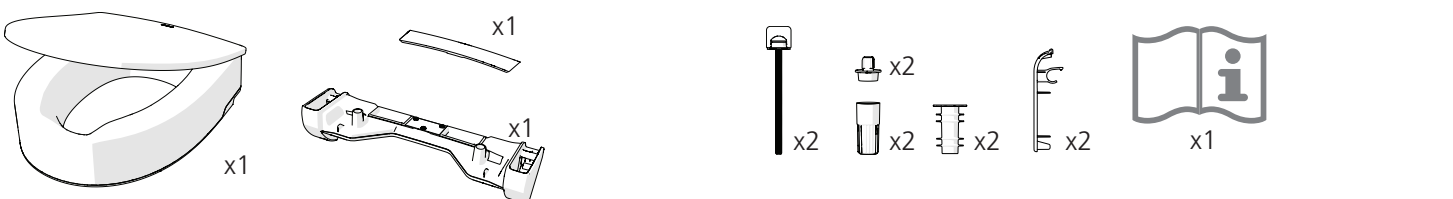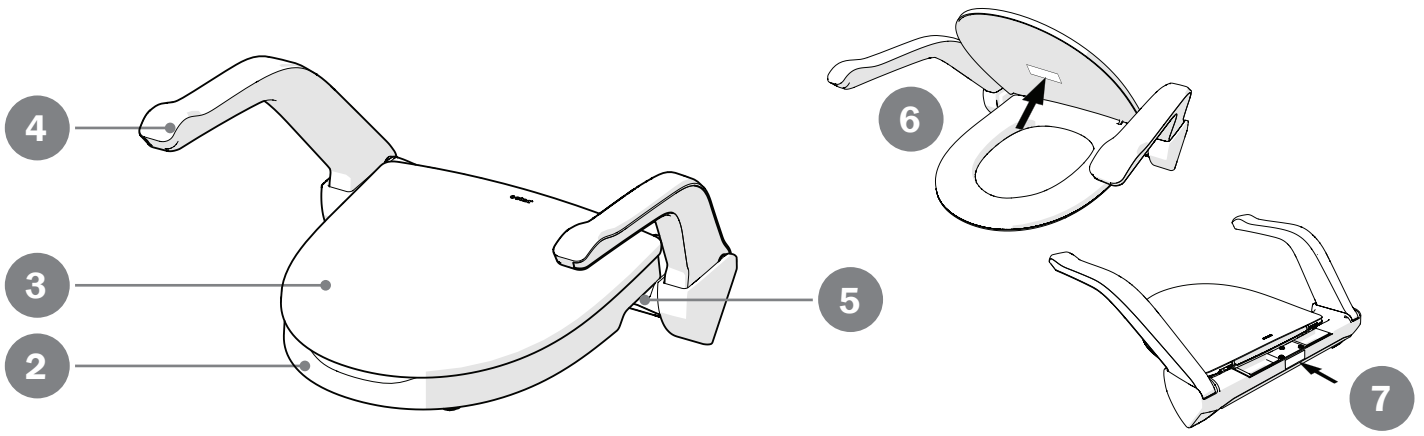8030 1519

## B

1		2		3		4		5	
6		7		8		9		10	
11		12							

## C

6

1 UDI **Bar code GS1-128**  
(01)XXXXXXXXXXXXXXXX(11)YYMMDD(21)XXXXX

2 XXXXXXXX  
XXXXXXXXXX

3 REF XXXXXXXXXXXX

4 SN XXXXXXXX

7 CE MD

8 Etac Supply Center AB  
Långatan 12  
SE-334 33 Anderstorp  
YYYY-MM-DD

5

# D

8030 1510	360 mm/ 400 mm	215 mm/ 285 mm	210 mm/ 275 mm	20 mm	480 mm	570 mm	215/245/285 mm
8030 1511	360 mm/ 400 mm	230 mm/ 360 mm	210 mm/ 275 mm	60 mm	480 mm	570 mm	175/205/245 mm
8030 1512	360 mm/ 400 mm	230 mm/ 360 mm	210 mm/ 275 mm	100 mm	480 mm	570 mm	135/165/205 mm
8030 1513	360 mm/ 400 mm	230 mm/ 360 mm	210 mm/ 275 mm	60 mm	-	-	-
8030 1514	360 mm/ 400 mm	230 mm/ 360 mm	210 mm/ 275 mm	100 mm	-	-	-

8030 1510	400 mm	c/c 123-203 mm	3.2 kg	150 kg (200 kg*)	50%
8030 1511	400 mm	c/c 123-203 mm	3.4 kg	150 kg (200 kg*)	50%
8030 1512	400 mm	c/c 123-203 mm	3.7 kg	150 kg (200 kg*)	50%
8030 1513	400 mm	c/c 123-203 mm	2.2 kg	150 kg	-
8030 1514	400 mm	c/c 123-203 mm	2.4 kg	150 kg	-

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### General information

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

### Device description

My-Loo Fixed is a raised toilet seat, available in three different heights. It is fixed mounted on the toilet and can be used with one, two or without arm supports. The arm supports are individually foldable.

### Intended purpose

My-Loo Fixed (hereafter also referred to as "the device" or "the product"), is a medical device intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to facilitate access to the toilet for people with restricted functional ability. The raised seat and/or arm supports intend to provide stability and support when sitting down and standing up before and after performing toilet needs.

### Intended user Group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more.

Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

### Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

### Intended application

The device is intended for short-term and long-term use and can be applied several times per day. The device is intended to be used in contact with intact skin.

The device is intended for refurbishment and reuse.

### Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see [www.etac.com](http://www.etac.com).

### Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

### Contraindications

There are no known contraindications.

**Warnings**

 A defective device must not be used. Risk of injury.

 Do not stand on the device. Risk of injury.

 Arm supports may only be used in the lowered position.

 The toilet must be securely anchored to the floor.

 Ignoring the instructions for use may result in injury.

**Declaration of conformity**

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

**Pre-purchase information**

Pre-purchase information can be found at [www.etac.com](http://www.etac.com).

**Adverse event**

In case of an adverse event occurred in relation to use of the device, it must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

**Customization**

is everything that goes beyond the manual's instructions and settings. Wheelchair specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

**Device description, delivery content.. [Figure A](#)**

1. Delivery content
2. Seat raiser
3. Toilet lid
4. Arm supports
5. Attachment
6. Product label\*
7. Item no label

**Symbols, labels ..... [Figure B](#)**

*Symbols in this manual:*

1. Warning, precaution or limitation.
2. Useful advice and tips.
3. Material for recycling.
4. Showing option 1 of two possible options.
5. Showing option 2 of two possible options.

*Symbols on the device:*

6. The commode must be securely anchored.
7. Maximum user weight (see technical data)
- 8-9. Cleaning (see maintenance)
10. Read instructions for use
11. Each arm support is designed to withstand a maximum load of half the user weight
12. Refer to user manual (blue and white).

**Explanation item no label: ..... [Figure C](#)**

1. Product name
2. Product description
3. Serial number
4. Item number
5. Manufacturer and date of manufacture
6. Barcode acc. to GS1-128 GTIN-14 and serial number
7. CE marked
8. Medical device

\*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

**Technical data..... [Figure D](#)****Assembly ..... [Figure E](#)****Dismantling arm supports..... [Figure F](#)****Mounting fixed arm supports..... [Figure G](#)****Maintenance**

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at

max. 85 °C if so required.

Constituent materials are corrosion-resistant. 5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

**Storage and disposal**

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

To be recycled according to national regulations.

**Accessories ..... [Figure H](#)**

1. Toilet paper holder
2. Storage basket
3. Supporting legs
4. Soft seat
5. Top-mounted screw kit (incl supporting legs, see section 3). Only in combination with arm supports and supporting legs
6. Arm supports

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**Etac Ltd.**
**Promefa AG**