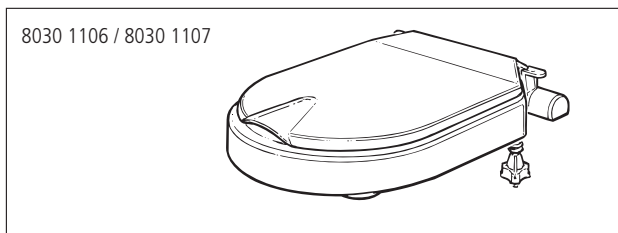
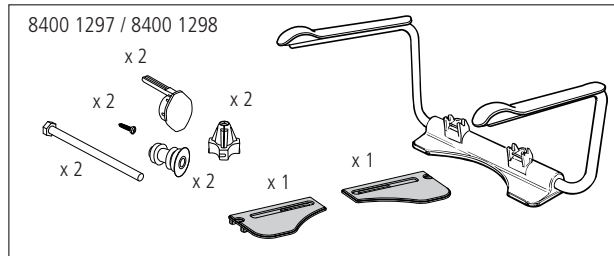
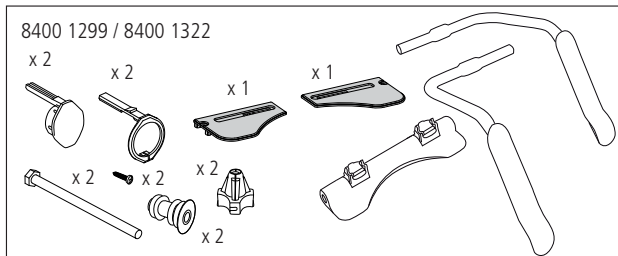
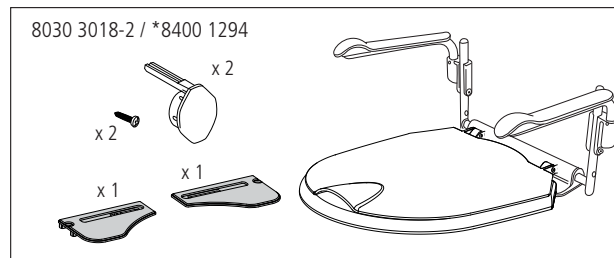
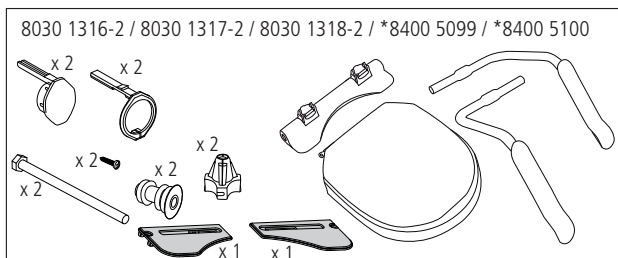
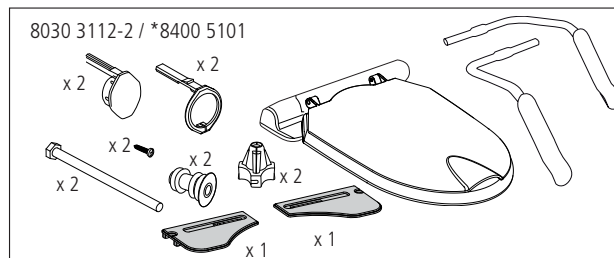
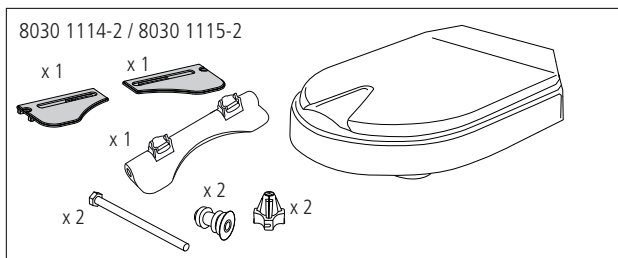
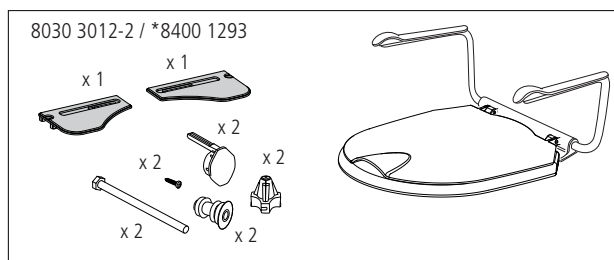
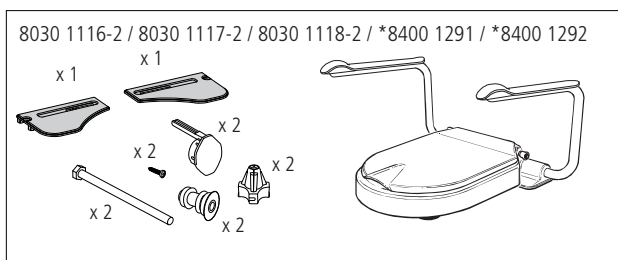
Hi-Loo/Supporter

Manual

etac®

74949H 2012-10-10

Leveransomfattning / Innhold i leveransen / Indeholdt i leveringene / Contenu de la livraison / Omvang van de levering / Toimituksen laajuus / Scope of delivery / Lieferumfang / La forniture include / Volumen de entrega

*Reservdel, endast armstöd / *Reservdel, kun armlene / Reservdel, kun armlæn / *Pièce détachée, repose-bras uniquement / *Reserveonderdeel, alleen armlenu-ning / *Varaosaja, vain käsinoja / *Spare part, only armrest / *Ersatzteil, nur Armlehne / *Ricambio, solo bracciolo / *Piezas de recambio, solo reposabrazos

Avsedd användning / Tiltenkt bruk / Beregnet anvendelse / Usage / Beoogd gebruik / Käyttötarkoitus / Intended use / Verwendungszweck / Uso previsto / Uso previsto

SE

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GB

Hi-Loo is a range of ergonomic toilet seat raisers that facilitates sitting down and getting up from the toilet. Hi-Loo can be used with or without armrests. Armrests provide greater security when seated and support when you sit down/get up from the toilet. The armrests can be flipped up, but can also be fixed in the lowered position.

Supporter is a range of toilet armrests that facilitates sitting down and getting up from the toilet. The armrests provide greater security when seated and support when you sit down/get up from the toilet. The armrests can be flipped up, but can also be fixed in the lowered position.

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Warranty: The product is covered by a five-year warranty.

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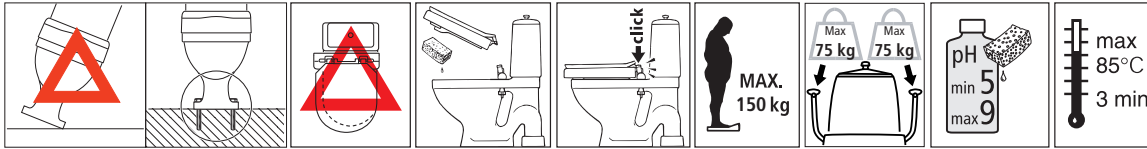
A defective product must not be used.

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Symbols:

- (GB)** The toilet chair must be securely fixed in place / The product must be fitted at the centre of the toilet chair / The seat ring can be removed when cleaning. Make sure it clicks in place completely when refitting / Maximum user weight 150 kg* / The armrests have each been tested and approved for half of the maximum user weight. / Clean the product with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85°C if so required.

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