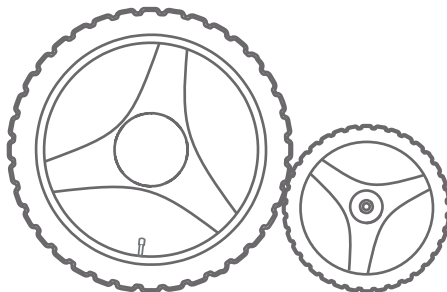
EN - Installing the air tires wheels
NL - Monteren van de luchtbandenwielen
DE - Montage der lufträder
FR - Installation des roues pneumatiques

**TOOLS
NEEDED**

Ring spanner size 19
Allen key size 6
Allen key size 5
Flat head screwdriver
Open-end wrench size 7
Combination pliers

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STEP 1: To remove the old front wheels, loosen up the axle of the front wheel using allen key size 6. If you have a Rollz rollator made before 2017, you will need allen key size 5. Take the wheel off by pulling it out.

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STEP 2: Place the new air tire in the fork. Use the axle from the old wheel to secure it in the fork. Repeat step 1 and 2 to replace the other front wheel.

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STEP 3: To remove the old rear wheels, first loosen up the screw that is holding the brake cable.

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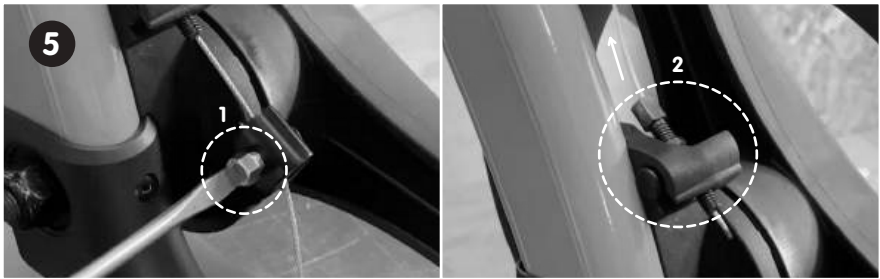
STEP 4: Pull the cable cap off sharply using combination pliers. If you don't succeed, try using a bolt cutter or a cable cutter to cut the cap off. However, this may result in split ends making the application of the new cap later more difficult.

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STEP 5: Unscrew the cable clamp using the flat head screwdriver and the open-end wrench (1). Remove the cable out of the cable clamp and from the adjustment screw (2).

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STEP 6: Approach the wheel from the lateral side and unscrew the screw connecting the drumbrake to the frame. Use allen key 5 for this.

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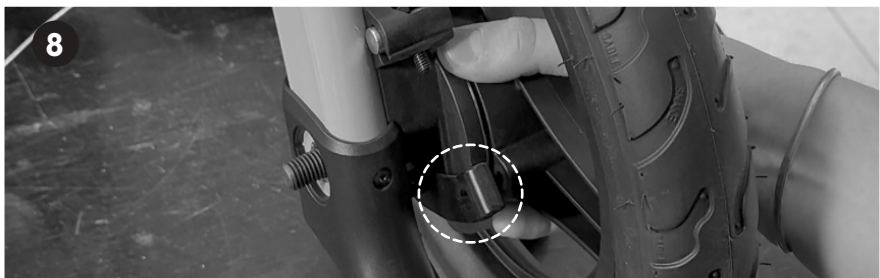
STEP 7: Unscrew the nut on the inside of the wheel using ring spanner size 19. Remove the wheel by pulling outwards.

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STEP 8: Place the new air tire. The left and the right wheel are mirrored, and positioned opposite to each other. Ensure the brake lever is pointing toward the rear when mounting the new air tire.

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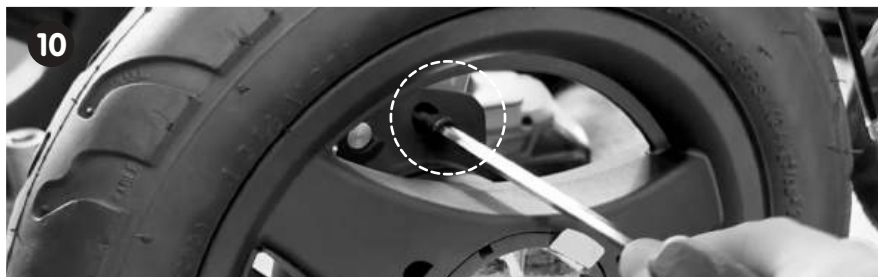
STEP 9: Place the ring on the screw (1) and secure it with the nut (2). Do not fasten it yet!

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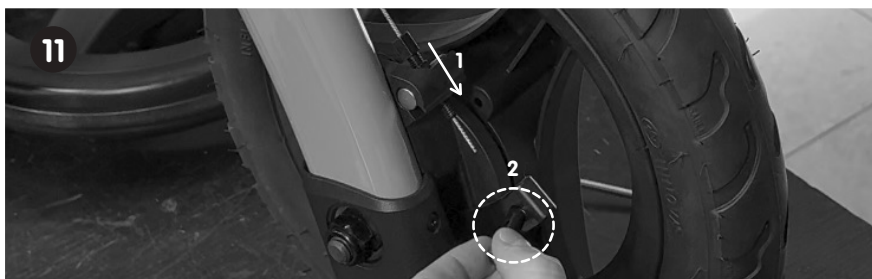
STEP 10: Reattach the screw that connects the drum brake to the frame from the lateral side of the wheel. Use allen key 5 to tighten it. Now you can fasten the nut from step 10 completely.

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STEP 11: Insert the brake cable into the adjustment screw (1), then secure it with the cable clamp (2).

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STEP 12: Insert the brake cable into the cable clamp (1) and pull it tight with pliers (this will raise the brake lever). Then, use a flathead screwdriver to securely tighten the cable clamp (2).

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STEP 13: Adjust the braking force with the adjustment screw. The brakes should be tight enough that the wheels no longer move when on the parking brake. The parking brake should be easy to release. To increase the braking force, tighten the screw upward. If the tension is too high, the parking brake will no longer engage properly. Adjust carefully to achieve the desired braking performance.

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STEP 14: Place the new cable cap on the brake cable and squeeze it in place with the combination pliers. Repeat steps 3 to 15 to replace the other rear wheel and perform a safety check.

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SAFETY CHECK | VEILIGHEIDSCONTROLE | SICHERHEITSPRÜFUNG | CONTRÔLE DE SÉCURITÉ

EN

1. Make sure all nuts, bolts, and screws are secure.
2. Make sure the wheels turn freely when the brakes are not engaged.
3. Walk around slowly and make sure the wheels come to a full stop when the brakes engage.
4. Make sure the parking brake (grey lever) works properly. The wheels should not turn when the parking brake is engaged.
5. Check if the parking brakes disengage properly by pulling on the grey levers.

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EN - If you have a Rollz Fit attached to the rollator's wheel, scan the QR code for instructions on how to remove it and install it on the air tires.

NL - If you have a Rollz Fit attached to the rollator's wheel, scan the QR code for instructions on how to remove it and install it on the air tires. This versatile 3-in-1 tool keeps the user active and independent regardless of mobility.

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