Rollz Fit

EN - Module and app guide

NL - Gids van de module en app

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For manuals in other languages,
please check **www.rollz.com**.

MODULE AND APP GUIDE

rollz fit

EN


The newest version of this manual can be found on our website and in the settings area of the Rollz app.

BEFORE YOU START


This guide is an addition to the Rollz Motion user manual. You can set up and use the Rollz Fit via the app after installing the device if it is not yet installed already.

WHAT'S IN THE BOX

Your Rollz Fit includes: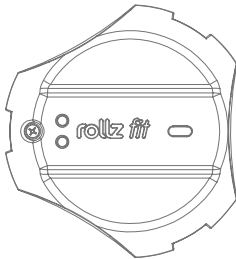

Rollz Fit

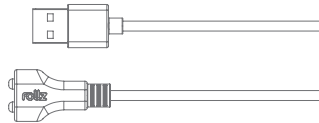Magnetic Charging Cable

TECHNICAL DETAILS OF THE MODULE

Operating temperature: -5°C to +45°C / 23 F - 113 F

Rated voltage and current: 5V DC - 0,5A

Battery capacity: 1200mAh

WARNING

The Rollz Fit contains electronic components. Do not submerge the Rollz Fit in water.

INTRODUCING ROLLZ FIT

Welcome to Rollz Fit, your go-to virtual activity buddy, tailor-made for rollator users. We're here to cheer you on as you embark on your journey to a more active and healthier lifestyle.

Our clever sensor sits snugly within the wheel cap of your Rollz rollator, ready to tag along on your daily adventures. It keeps you in the loop about the distances you cover each day, making every step count!

Thanks to the Bluetooth feature, syncing with the Rollz app is a breeze, unlocking a trove of perks for you. Inside the app, you'll discover a wealth of insights into your activity levels, the freedom to set personalized fitness goals, and the excitement of unlocking achievements. Plus, you can easily keep tabs on your progress over time, celebrating every milestone along the way. Get ready to see your essential stats - like distance walked, calories burned, and active minutes - front and center, keeping you motivated and moving towards your fitness goals!

Ready to dive in? Just scan the QR code and uncover all the fantastic features waiting for you with Rollz Fit.

STAY ACTIVE WITH ROLLZ APP

The Rollz app can be downloaded for free on your smartphone or tablet from the Play Store (for Android) or the App Store (for iOS). Search for "Rollz".


Rollz

ATTACHING THE ROLLZ FIT

If the Rollz Fit has not been installed yet, please follow the steps below. The Rollz Fit should be placed at the wheelcap on the right side (from the user's perspective). It can be attached after removing the original wheelcap.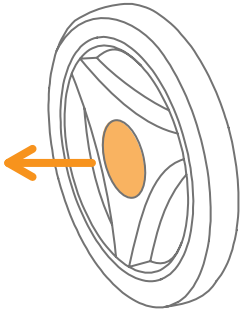


Removal

Removing the wheelcap can be done using a prying tool:


- Use the prying tool on the holes of the wheelcap to lift it as far as possible
- Repeat this process with the other holes
- Once the cap is lifted enough, remove it by pulling it off

Attaching

Align the Rollz Fit with the rim and press it against the spokes until the flanges are snugly pressed against them. The Rollz Fit will extend beyond the level of the wheelcap.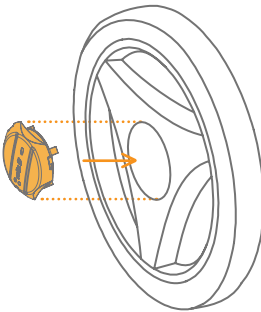

Make sure the Rollz Fit pins are connected in the U-shaped holes.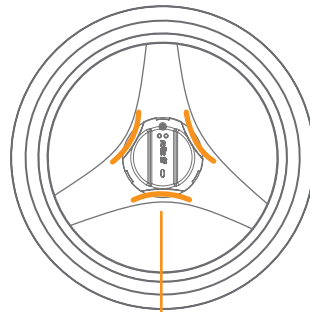

Make sure the shapes are aligned.

CHARGING THE ROLLZ FIT


Before starting the device, make sure the Rollz Fit is charged.



WARNING

Use only the supplied magnetic charging cable to charge the module.

1. To charge, use the included USB Magnetic Charging Cable. The cable has 2 magnetic heads that securely dock onto your device, facilitating fast recharging.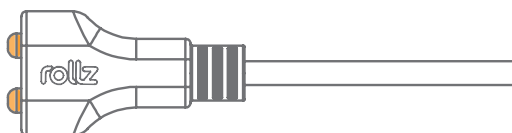

2. Connect the USB Magnetic Charging Cable to the device.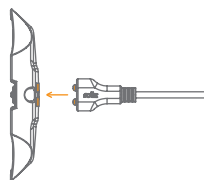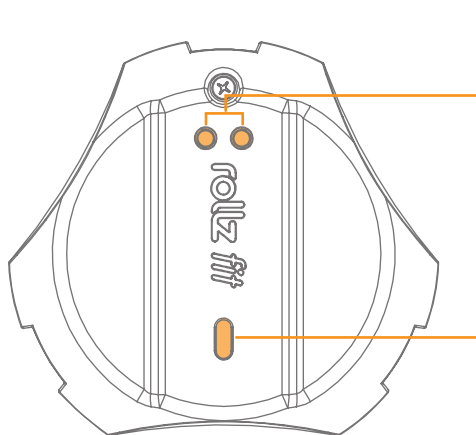

The 2 magnetic heads of the charging cable magnetically snaps to the Rollz Fit.

- Charge the device only with a 5V and 1A adapter
- The device will display a red light while charging
- The light will become green when fully charged

Battery life depends on usage. The battery can be charged at any time. A fully depleted battery takes 4 hours to recharge.

NOTE

When the battery is under 10% or empty, the LED flashes briefly when the walker is in motion.

'blink'


TURNING ON AND WAKE THE ROLLZ FIT

Before starting the device, make sure the Rollz Fit is charged.

To turn on the Rollz Fit when it's in sleep mode, simply rotate the wheel by moving the rollator forward or backward. This action will automatically activate the device (indicated by a single flash of the blue light).

When the Rollz fit is in operation a blue light will flash once every 3 seconds.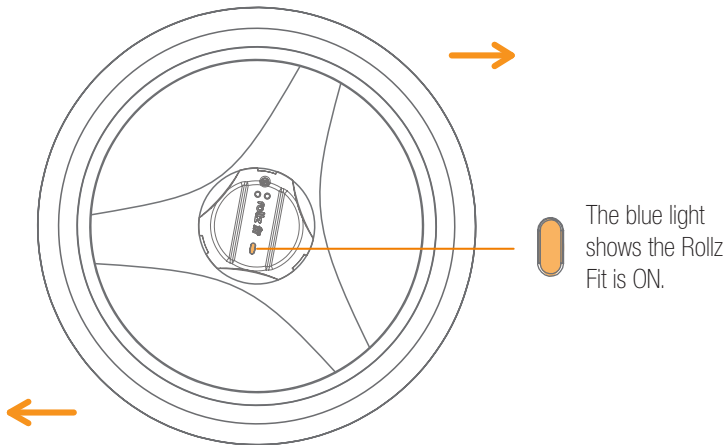


If the Rollz Fit is left without any movement for a while, it will automatically enter SLEEP mode. When you resume walking with the Rollz Motion, the Rollz Fit will automatically become active again.

CONNECTING THE ROLLZ FIT

Turn on the device by moving the rollator and open the Rollz app to connect.

When opening the app, a few steps have to be taken to connect to the Rollz Fit. These steps only need to be taken the first time. Afterward, the Rollz Fit homepage will open directly, and the app will search for and connect to the Rollz Fit automatically.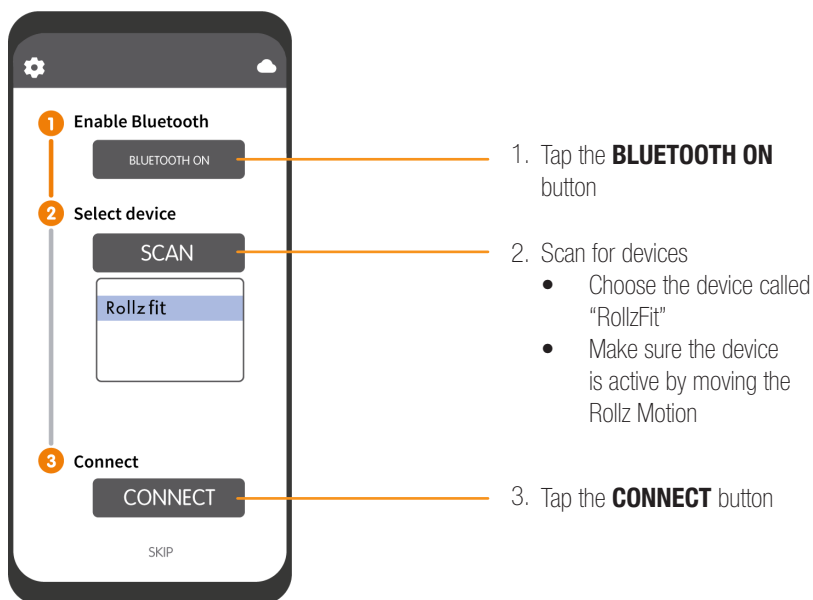

When the Rollz Fit device is successfully connected, the Rollz app will display the connected symbol  in the upper right corner. At the same time, the Rollz Fit will also indicate the connection with a blue fading light.

Go to the app settings  to set up profile; the preferred language or to find our contact information and website.


NOTE

The Rollz Fit only needs to be connected to the Rollz app once. The device will automatically connect to the app when it is nearby. Ensure the device is turned on by moving the Rollz Motion.

USING THE ROLLZ FIT WITH THE APP

Basic functionality is described in these guidelines; therefore, some of the drawings and information might appear different in reality as we continuously strive to improve the user experience.

The Home screen  shows the daily activity goal. Scroll down in the Rollz app to see your daily stats.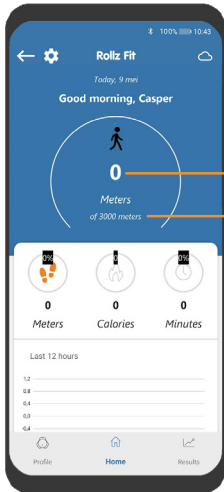

Progress measured by Rollz Fit.

Goal set by user.

Daily stats of distance, calories and minutes.


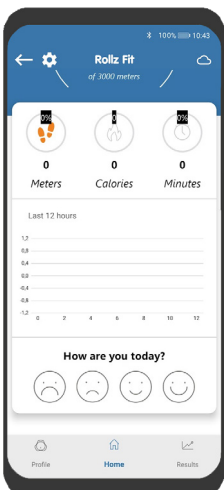
Chart data for daily progress.

Rate your personal fit feeling to compare it with the results.

ADJUSTING THE ROLLZ FIT SETTINGS

Manage the basic settings in the Profile page  .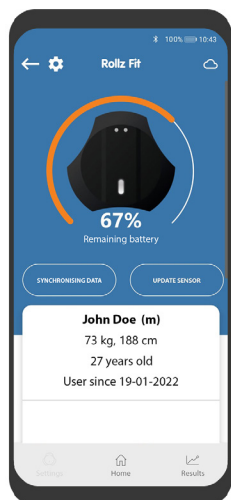

Rollz Fit battery percentage.

Synch data from Rollz Fit device & Sensor update.


The update button is only visible when an update is available.

User information.

CHECKING THE RESULTS

Check the overall progress in Results .

The calendar displays the days you have been active. Select a day to view the activities. When a goal is set, colored markings indicate the days the goal is met.

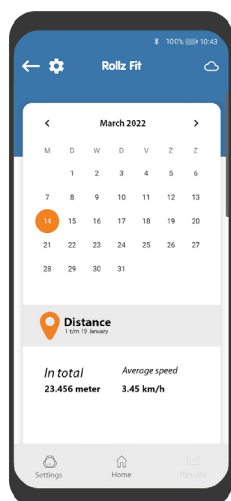Progress overview.

Measurements.

LED SIGNALS

The LED indicate the operating state of the Rollz Fit.

LED	Operating state
The blue LED is OFF	The Rollz Fit is in SLEEP mode.
The blue LED blinks once in 3 seconds	The Rollz Fit is ON.
The blue LED blinks fast when starting	The Rollz Fit battery is below 10% and needs to be charged.
The blue LED is fading	The Rollz Fit is connected to the Rollz app via Bluetooth.
The red LED is fading	The Rollz Fit is charging.
The Green LED is ON	The Rollz Fit is fully charged.

PRIVACY PROTECTION

Rollz International treats your privacy with care. By using the Rollz app, you agree that some anonymous data is collected to enhance the app experience. This data will not be used for sales purposes or shared with third parties.

WARRANTY

The Rollz Fit accessory is covered by the following warranty periods:

1. The Rollz Fit is under warranty for a period of 24 months.
2. The battery in the Rollz Fit device is fully warranted for 12 months.

USER TIPS

Tips for the optimal use of your Rollz Motion can be found at www.rollz.com.

DISPOSAL AND RECYCLING INFORMATION FOR ROLLZ FIT

Your Rollz Fit accessory should not be disposed of with household waste. Please dispose of your Rollz Fit in accordance with local environmental laws and guidelines.

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