
User manual

Molift RgoSling Fabric Stretcher

molift[®]
by Etac

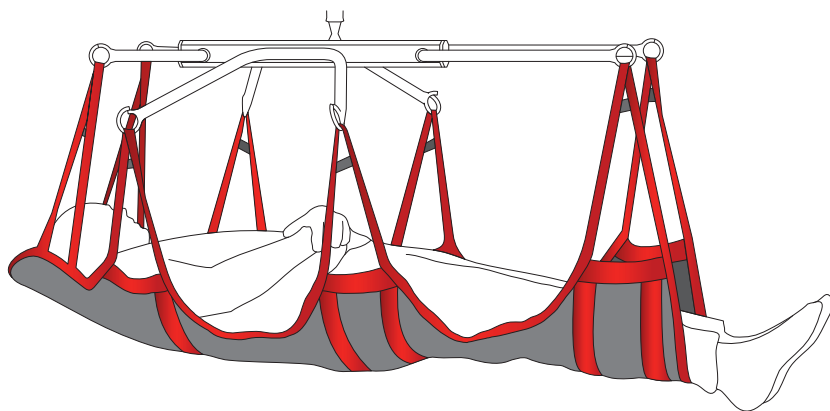BM40799 - 2024-06-25 - Rev. 5.0

www.etac.com

	Symbols	4
	Device overview	8
	General	10
	English	10
	Svenska	14
	Norsk	18
	Dansk	22
	Suomi	26
	Deutsch	30
	Nederlands	34
	Français	38
	Italiano	42
	Español	46
	Handling	50
	Lying application	51
	Lying removal	53
	Warnings	54
	Information	55
	Combination list	57
	Sling measurements	57

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
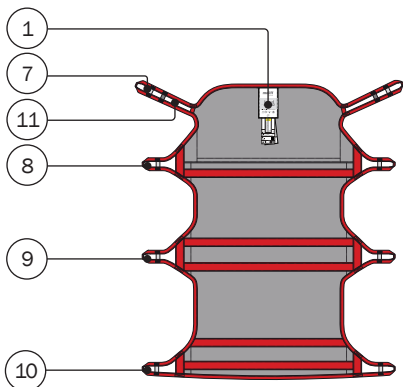
Molift RgoSling Fabric Stretcher (Outside)

Product label

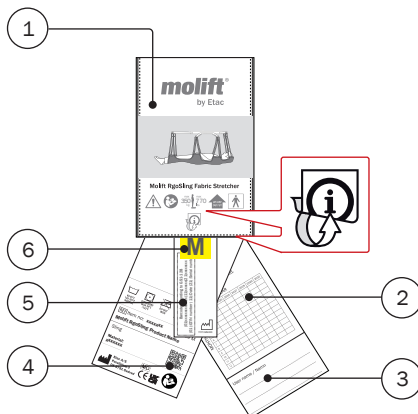1

EN: Product label
SV: Produktetikett
NO: Produktetikett
DA: Produktmærkat
FI: Tuotetarra
DE: Produktetikett
NL: Productlabel
FR: Étiquette du produit
IT: Etichetta prodotto
ES: Etichetta prodotto

2

EN: Periodic inspection
SV: Periodisk inspektion
NO: Periodisk inspeksjon
DA: Periodisk inspektion
FI: Määräaikaistarkastus
DE: Regelmäßige Inspektion
NL: Periodieke inspectie
FR: Inspection périodique
IT: Ispezione periodica
ES: Inspección periódica

3

EN: User name
SV: Brukarnamn
NO: Brukernavn
DA: Brugernavn
FI: Käyttäjän nimi
DE: Benutzername
NL: Gebruikersnaam
FR: Nom de l'utilisateur
IT: Nome utente
ES: Nombre de usuario

4

EN: QR-code
SV: QR-kod
NO: QR-kode
DA: QR-kode
FI: QR-koodi
DE: QR-Code
NL: QR-code
FR: Code QR
IT: Codice QR
ES: Código QR

5

EN: (01) EAN-code,
(11) date,
(21) Serial number
SV: (01) EAN-kod,
(11) datum,
(21) Serienummer
NO: (01) EAN-kode,
(11) dato,
(21) Serienummer
DA: (01) EAN-kode,
(11) dato,
(21) Serienummer
FI: (01) EAN-koodi,
(11) Päivämäärä,
(21) Sarjanumero
DE: (01) EAN-code,
(11) Datum,
(21) Serial number
NL: (01) EAN-code,
(11) datum,
(21) Serial number
FR: (01) Code-EAN,
(11) date,
(21) Numéro de série
IT: (01) Codice EAN,
(11) data,
(21) Numero di serie
ES: (01) Código EAN,
(11) fecha,
(21) Número de serie

6

EN: Size label
SV: Storleksetikett
NO: Størrelsesetikett
DA: Størrelsesmærkat
FI: Kokotarra
DE: Größenangabe
NL: Maatlabel
FR: Étiquette de taille
IT: Etichetta taglia
ES: Etiqueta de tamaño

7

EN: Head strap loops
SV: Bandöglor huvud
NO: Løftestropp hode
DA: Hovedstrop
FI: Pään kohdalla olevat nostolenkit
DE: Hebeschlaufen im Kopfbereich
NL: Bandlussen hoofd
FR: Boucles des sangles de tête
IT: Passanti per cinghia capo
ES: Lazos de las correas de la cabeza

8

EN: Shoulder strap loops
SV: Bandöglor axlar
NO: Løftestropp skulder
DA: Skulderstrop
FI: Olkapäiden kohdalla olevat nostolenkit
DE: Hebeschlaufen im Schulterbereich
NL: Bandlussen schouder
FR: Boucles des sangles d'épaules
IT: Passanti per cinghia spalle
ES: Lazos de las correas de los hombros

9

EN: Waist strap loops
SV: Bandöglor midja
NO: Løftestropp midje
DA: Taljestrop
FI: Vyötärön kohdalla olevat nostolenkit
DE: Hebeschlaufen im Taillenbereich
NL: Bandlussen taille
FR: Boucles des sangles de taille
IT: Passanti per cinghia vita
ES: Lazos de las correas de la cintura

10

EN: Leg strap loops
SV: Bandöglor ben
NO: Løftestropp ben
DA: Benstrop
FI: Jalkojen kohdalla olevat nostolenkit
DE: Hebeschlaufen im Beinbereich
NL: Bandlussen been
FR: Boucles des sangles des jambes
IT: Passanti per cinghia gambe
ES: Lazos de las correas de las piernas

11

EN: Loop connections
SV: Ögleanslutning
NO: Løftestropp
DA: Løftestroppe
FI: Lenkkiliitintä
DE: Schlaufenverbindung
NL: Lusaansluiting
FR: Fixation par boucle
IT: Collegamento ad asola
ES: Conexión al nudo

General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit www.etac.com for all relevant product documentation, for example the user manual, periodic inspection document and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the product label for further information and instructions.

Copyright

This document is the property of Etac A/S. The content shall not be modified in whole or part without prior approval of Etac A/S.

Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult www.etac.com to find the latest version of the documentation.

CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 10535:2021 Assistive products – Hoists for the transfer of persons – Requirements and test methods.

Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

Pre-sales information

Please visit www.etac.com.

Device description

Etac supplies a wide selection of slings for different types of transfers. The Molift RgoSling series is designed to provide high comfort for users by evenly distributing weight and pressure. Molift RgoSlings are available in different sizes, models and materials. See www.etac.com for a complete overview of possible variants.

Molift RgoSling Fabric Stretcher can be used for supine transfers between two horizontal surfaces. The stretcher is designed to be combined with an 8-point sling bar.

Benefits

The stretcher is designed to provide comfort and full support for users requiring a stable, recumbent position, for example comatose or sedated clients.

The loop connections give multiple options for attachment of the stretcher.

Materials

Fabric: Polyester

Padding: Polyethylene

Introduction

The person referred to as “the user” is the person who is lying in the product. “The carer” is the person who applies the product.

Intended purpose

The stretcher is an assistive device intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed for an individual lacking the ability to transfer themselves in lying position between two beds or similar due to reduced mobility or physical strength.

Intended user

The device can be used for children, adults and elderly people with a need to be lifted in lying position.

Intended environment

For indoor use in acute care, long term care and home care.

Indications

For users who are in need of support to be lifted in lying position for a short time; e.g. people with limited ability to move and who require body support during transfers. Not an exhaustive list.

The user group for the device is based on individual health and mobility function, and not on a specific diagnosis or age.

Contraindications

There are no known contraindications.

Precautions

Make sure the stretcher fits the user and the sling bar.

This product requires an 8-point sling bar.

Training requirements

Only carers who have read and understood the user manual are allowed to use the device.

Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warnings symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

Never leave a user unattended when hoisted.

Molift slings shall only be used to hoist persons. Never use the sling to hoist objects of any kind.

Risk assessment

Do a risk assessment of the sling, the individual user and the intended transfer, and document your analysis. You as the carer are responsible for the safety of the user.

Make a decision on whether one or more carers are required.

Plan the transfer in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

Before using the sling, mark the date of first use on the periodic inspection label.

Slings

Always check – before each use:

- that the user has the required ability for the sling.
- that the model, size and fabric are suitable for the user and the transfer situation.
- that the sling is suitable for the hoist and the sling bar.
- that the sling does not have visible damage or frays to seams, fabric, straps and loops.
- that the loop connections are correctly attached to the sling bar hooks before hoisting the user.
- that the wheels on the wheelchair or bed are locked during the transfer. If using a mobile hoist the wheels on the hoist must not be locked.

If the inspection reveals any safety risks, the sling must immediately be taken out of service and be disposed of.

Never use a faulty or damaged sling as it can break and cause personal injury. Destroy and discard damaged and old slings.

Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/careers who are allergic to any of the materials are recommend not to use the device.

Accessories

User manuals supplied with accessories must be read and understood before use. This includes any limitations in use and combinations. Information about accessories can be found on www.etac.com.

Combinations

This device can be combined with Molift sling bars and hoists. See the combination list on page 57.

Statement on sling and hoist compatibility for other manufactures of hoists can be found on www.etac.com.

When combining the device with other devices, the combination must not change the intended purpose of the devices or modify the devices in such a way that compliance with the applicable requirements may be affected. The person/company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

If safe working load (SWL) differs between hoist, sling bar and sling, the lowest load should always be used.

Troubleshooting

Problem	Action
The user's shoulders and hips do not fit the dimensions of the two centre straps.	Try another size.
The space is too limited for the user's shoulders and hips.	Try a larger size.
The straps are not loaded at the same time.	Make sure the straps are attached symmetrically.
The user is not hoisted in a balanced position.	Make sure hips and shoulders are placed in line with the two centre straps. Make sure the centre of the stretcher is at the mid-line of the user.

Service, cleaning and maintenance

Maintenance and periodic inspections of the device are wholly the responsibility of the owner of the device. Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

Expected lifetime

The device has an expected lifetime of 1 to 5 years under normal use. The lifetime of the device may vary depending on usage frequency, loads and washing/drying frequency and conditions.

Washing and drying at high temperatures wear out the material faster.

Periodic inspection

Periodic inspection must be carried out at least every 6 months. More regular inspection may be required if the sling is used or washed heavily. See the periodic inspection label on the sling and the periodic inspection document at www.etac.com.

Cleaning

60-85°
140-185F

Max. 60°
/140F

Transportation, storage and disposal

Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Disposal

The device contains no hazardous substances that require special disposal. Dispose of the device according to the national provisions that apply.

Recycling

See the recycling instructions on www.etac.com, and follow the national provisions that apply.

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EN: Information

01. The stretcher should be centered on the midline of the user.
02. Make sure the straps are fastened symmetrically on left and right side.
03. Before performing the transfer – make sure that:
 - User's head is well supported by the stretcher.
 - All eight straps are securely fastened to the sling bar.
 - User's weight is horizontally balanced when hoisting.
04. Perform a safety stop when the straps are stretched tense. Check that all straps are correctly and symmetrically attached.

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**Etac Ltd.**